



## POCT Urine ACR EQA

### INTENDED USE

Weqas POCT Urine ACR EQA samples are for in-vitro diagnostic use as an external quality assessment material for testing of Urine Albumin, Creatinine and Albumin/Creatinine Ratio (ACR).

### SUMMARY

External Quality Assessment (EQA) or proficiency Testing (PT) is an essential part of providing quality laboratory diagnostic services, and participation in EQA is required for laboratory accreditation to ISO 15189 and ISO 17025. EQA is the inter laboratory comparison and performance evaluation that extends throughout all phases of the healthcare diagnostic testing cycle.

### PRODUCT DESCRIPTION

0.5 mL volume is supplied in sterile plastic tubes. The material has been prepared from sterile urine from healthy donors, with an antibiotic added to maintain sterility. The urine is dispensed and stored at -20°C until dispatch.

### STORAGE AND STABILITY

If you are unable to assay the samples immediately upon receipt, please store at 2-8°C until analysis. Samples stored at 2-8°C prior to analysis must be brought to ambient temperature (18-30°C) prior to use.

Samples should be stored at -20°C for long term storage.

### PROCEDURE

The samples should be treated the same as patient specimens and run in accordance with the instructions accompanying the test system being used.

1. Mix the sample well by gently inverting 5 to 6 times.
2. Wear gloves and handle the sample as a normal patient sample.
3. Safely dispose of excess sample in accordance with local waste policy guidelines.

**Always wear gloves to avoid contamination.**

### LIMITATIONS OF PROCEDURE

The POCT Urine ACR EQA samples should be analysed according to the instructions within this document. If there is evidence of microbial contamination or excessive turbidity in the product, discard the vial.

The POCT Urine ACR EQA samples require storage as described in STORAGE AND STABILITY and handling as described in PROCEDURE.

Accurate and reproducible results are dependent upon properly functioning instruments and reagents and the use of correct procedures.

### ANALYTES COVERED

The Weqas POCT Urine ACR EQA samples cover a relevant pathological and analytical range.

Analyte
Albumin
Creatinine
Albumin / Creatinine Ratio

### ! CAUTION !

#### Human source material.

The base material is sterile urine from 'normal' volunteers.

**Although every effort is made to ensure that the material is free from any known infectious agent, the samples should be handled as for clinical specimens.**